



# Sleeves

## TECHNICAL INSTRUCTION



ZI de Sacuny  
118 Avenue Marcel Mérieux, 69530 BRIGNAIS  
Tel.+33 (0)4 78 56 97 00  
Fax +33 (0)4 78 56 01 63  
www.globald.com

This document aims to specify instruction of use of the STECO® sleeves or Master tubes. They are designed for pilot or full guided surgical guide's protocols as well as for pins and screws to fix the guide.

**Global D offers a STECO® range of sleeves or Master tubes adapted to surgical guides following pilot drilling or full guided drilling protocols as well as pins & screws to fix the guide.**

The surgical guide design is based on the practitioner implant planning. It is manufactured with the best precision possible by 3D printing at the practitioner dental clinic or at his prosthetic dental laboratory or any outsourced service facility. The sleeves are then placed or glued, if necessary, in the holes of the guides designed during the earlier planning phase.

Global D recommend setting in advance the 3D printer with the most adapted tolerances between the internal diameter of the guide's holes and the external diameter of the sleeves in order to get the best retention of each type of sleeves.

Find below the **full range of STECO® sleeves** or Master Tube distributed by **Global D** :

		Global D reference	STECO® reference
<b>G20 pilot drill surgical guide's sleeves</b>	<b>Pack of 10</b>	DMTS2.0L5	M.27.31D200L5
<b>G42 full guided surgical guide's sleeves</b>	<b>Pack of 5</b>	DMTS4.2L4	M.27.15.D420
<b>G52 full guided surgical guide's sleeves</b>	<b>Pack of 5</b>	DMTS5.2L4	M.27.15.D520
<b>Sleeves for pins</b>	<b>Pack of 10</b>	DMTS1.5L10	M.27.20.D150L10
<b>Sleeves for Graftek screws ref VA1.5KL11 /KL13 /KL15</b>	<b>Pack of 10</b>	DMTS1.5L6	M.27.03.D150L6



\* Sleeves (réf DMTS2.0L5 , DMTS4.2L4, DMTS5.2L4, DMDMTS1.5L6 et DMTS1.5L10) are medical devices manufactured and CE marked by STECO system technik GmbH &Co.

The products shown are Medical Devices and as such bear the CE mark in compliance with Directive 93/42/EEC. In France, these devices are not subject to the social security reimbursement scheme. The medical devices shown may not be available for sale in all countries. If you require any further information please contact the Global D sales department. Please read the instructions before use. If in doubt, please contact the Global D sales department. Our instructions for use are digital. Please refer to the QR code and the URL link shown on the device label. The leaflets are nevertheless available on request and without additional charge within 7 days. The request must be sent to the following address: [quality@globald.com](mailto:quality@globald.com)